

# Oxivir® Tb and Oxivir® Tb Wipes

## General Virucide, Bactericide, Tuberculocide, Fungicide, Sanitizer



---

**A ready-to-use cleaner disinfectant based on proprietary AHP® technology to deliver fast, effective cleaning performance.**

### Effective

- ▶ Bactericidal and \*Virucidal in one minute, kills HBV, HCV, HIV-1 (AIDS virus) in one minute, Tuberculocidal in five minutes and kills fungi in ten minutes
- ▶ Patented technology contains a high performance surfactant system that delivers faster and better cleaning results
- ▶ SKU #5388471 meets NFPA (National Fire Protection Association) and CMS (Centers for Medicare & Medicaid Services) requirements

### Environmentally Responsible

- ▶ No VOC's, no added fragrance, no NPE's
- ▶ The active ingredient breaks down to water and oxygen

### Easy-to-Use

- ▶ Use to clean most hard, nonporous environmental surfaces with confidence
- ▶ Non-irritating to skin and eyes and contains no substances which at the given concentration, are considered to be hazardous to health per OSHA standards

---

### Designed For

- ▶ Health Care
- ▶ Retail
- ▶ Government
- ▶ Lodging



# Use Overview

**Oxivir® Tb** is a patented one-step cleaner disinfectant for spray-and-wipe cleaning applications. **Oxivir® Tb Wipes** is a one-step cleaner disinfectant wipe.

- ▶ Pre-clean heavily soiled areas.
- ▶ Apply solution by cloth or disposable wipe to hard, non-porous environmental surfaces.
- ▶ All surfaces must remain wet for one-minute to kill bacteria, HIV-1, HBV and HCV.
- ▶ Use a five-minute contact time for Tb and a ten-minute contact time for fungi.
- ▶ Wipe surfaces dry, rinse or allow to air dry.
- ▶ **Note:** Rinsing is not necessary unless floors are to be coated with finish or restorer. All food contact surfaces must be rinsed with potable water. Do not use on glassware, utensils, or dishes. Not recommended for use on brass, copper, aluminum, bronze, granite, zinc or marble.



## Product Specifications

Description	
<b>Color/Form</b>	Colorless, liquid; Colorless, wipe
<b>EPA Reg. No.</b>	70627-56 (RTU) 70627-60 (Wipes)
<b>pH</b>	2.5–3.5
<b>Scent</b>	Characteristic
<b>Solubility</b>	Complete
<b>Shelf Life</b>	2 years (RTU), 3 years (Wipes)
<b>Specific Gravity</b>	1.01

## Hazard Rating

HMIS		NFPA
0	Health	0
0	Flammability	0
0	Reactivity	0

4=Very High; 3=High; 2=Moderate; 1=Slight; 0=Insignificant.

## Available Items

Product Code	Description/Package Size	Dilution
4277285	12 x 32 fl. oz. / 946 mL Flip Top Bottles Plus Two Trigger Sprayers	Ready-to-Use
5768705	4 x 32 oz./946 mL Flip Top Bottles Plus One Trigger Sprayer	Ready-to-Use
4599516	12 x 160 count 6 x 7 in. Wipes / 15.24 x 17.78 cm Canisters	Ready-to-Use
5388471	12 x 60 count 7 x 8 in. Wipes / 17.8 x 20.3 cm Canisters	Ready-to-Use
5499664	6 x 1 Unit (Wall Bracket for use with 60 count wipes)	
9P700798	1 Double Unit Non-Locking Stand	
9P700797	1 Double Unit Locking Stand	
9P700814	1 Plastic Locking Bracket (for use with 160 count wipes)	
9P700799	6 x 1 Unit Wire Bracket Locking on Plastic Wall Mount (for use with 160 count wipes)	
9P700800	6 x 1 Unit Wire Bracket Non-Locking on Plastic Wall Mount (for use with 160 count wipes)	

## Safety Reminder

Please make sure your employees read and understand the product label and Material Safety Data Sheet before using this product. The label contains directions for use, and both the label and MSDS contain hazard warnings, precautionary statements and first aid procedures. MSDS are available on-line at [www.diversey.com](http://www.diversey.com) or by calling 888 352 2249.

AHP and Design® are registered trademarks of Virox Technologies Inc. used under license.

For additional information, please contact us at +1 800 558 2332 or at [www.diversey.com](http://www.diversey.com)  
© 2012 Diversely, Inc., 8310 16th Street, Sturtevant, WI 53177-1964. All Rights Reserved. SPC729 (12/080)

**Diversely**  
for a cleaner, healthier future™